


Installation Guide

Tower Mount Bracket for DINAMAP® Mobility Workstation


Use to mount 75x75mm mounting pattern devices

Capsule Neuron with Mini Dock


Capsule Neuron 2

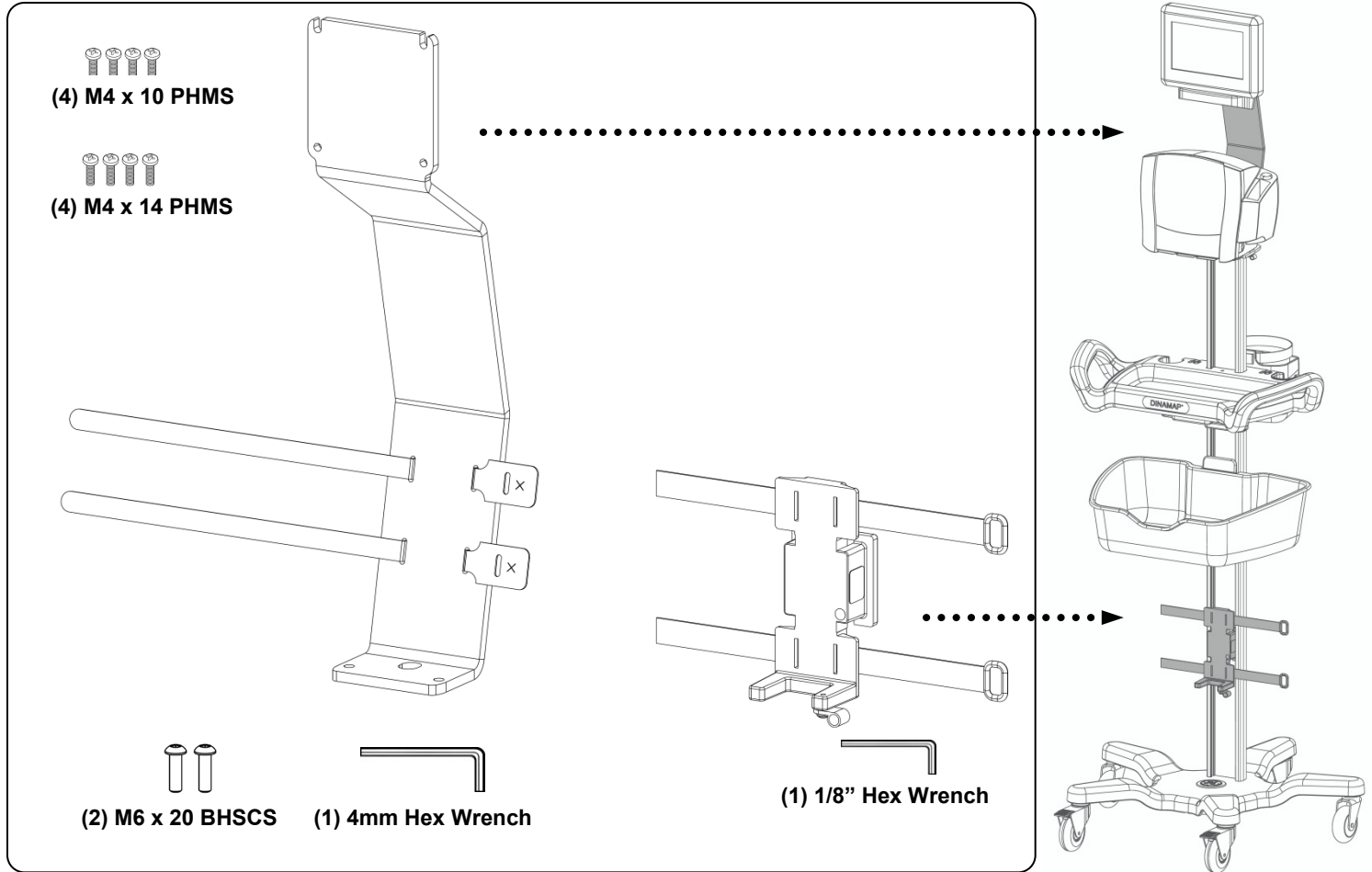

1

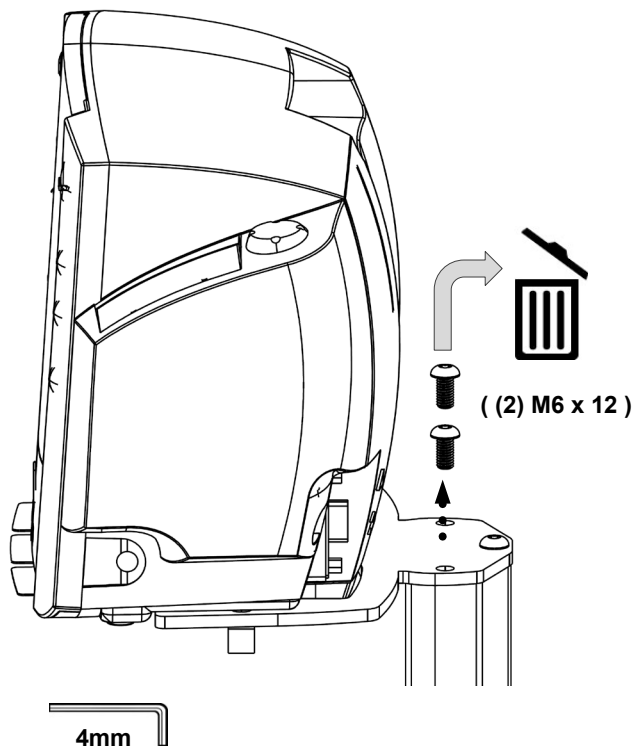


2

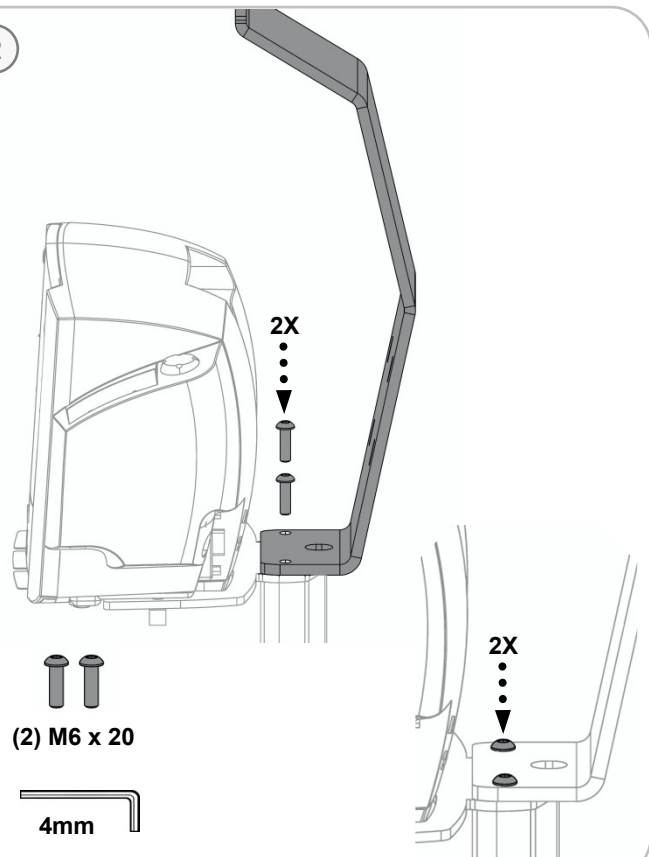

3



Capsule Neuron
with Mini Dock






(4) M4 x 14 PHMS

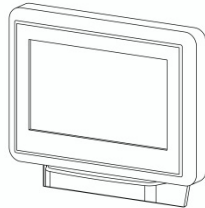

Capsule Neuron 2

(4) M4 x 10 PHMS

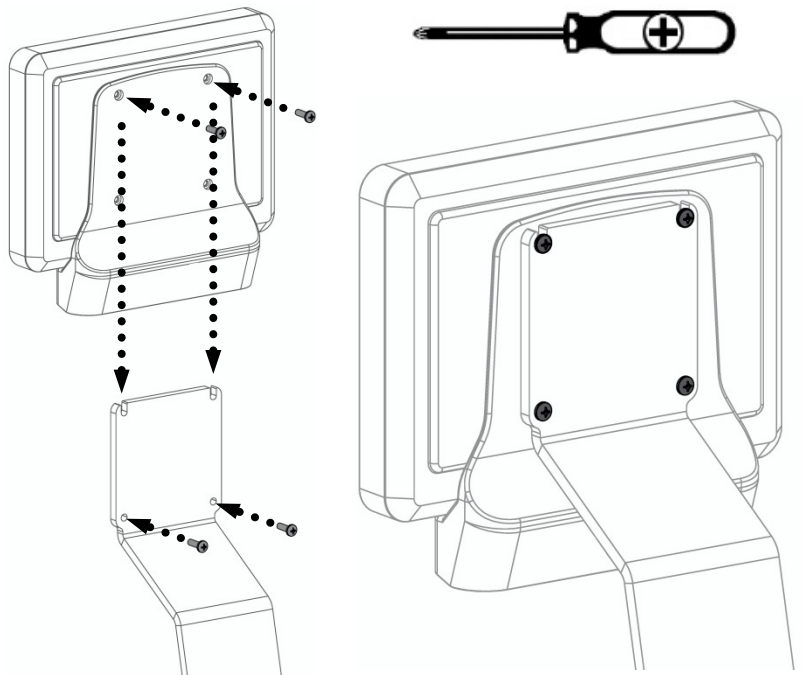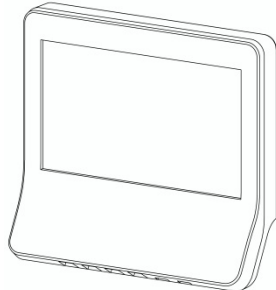

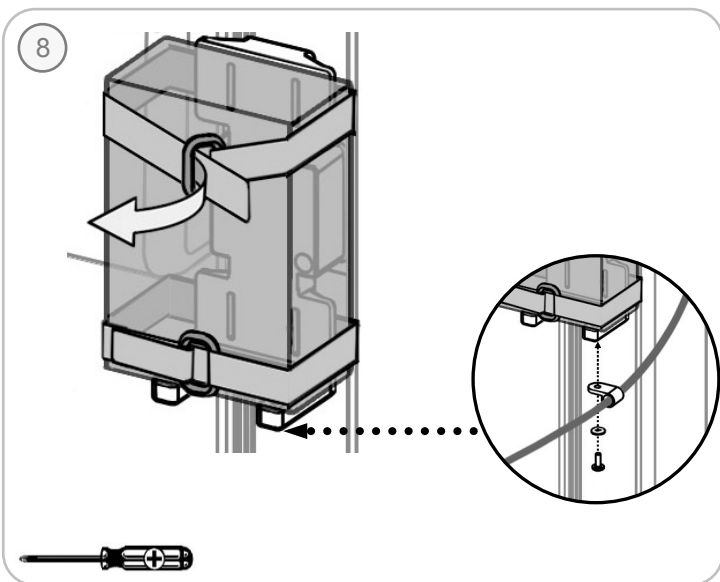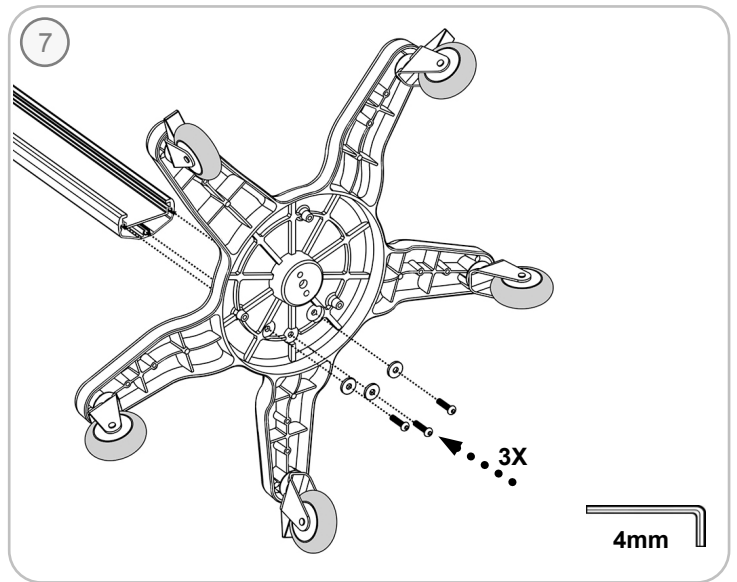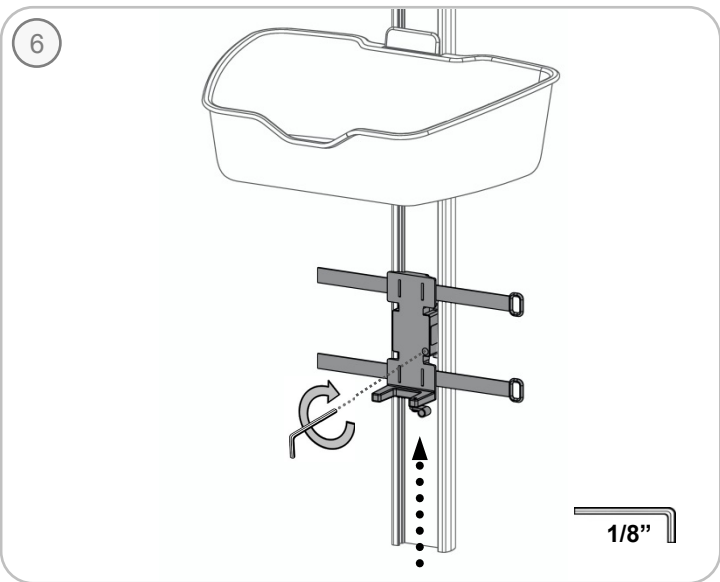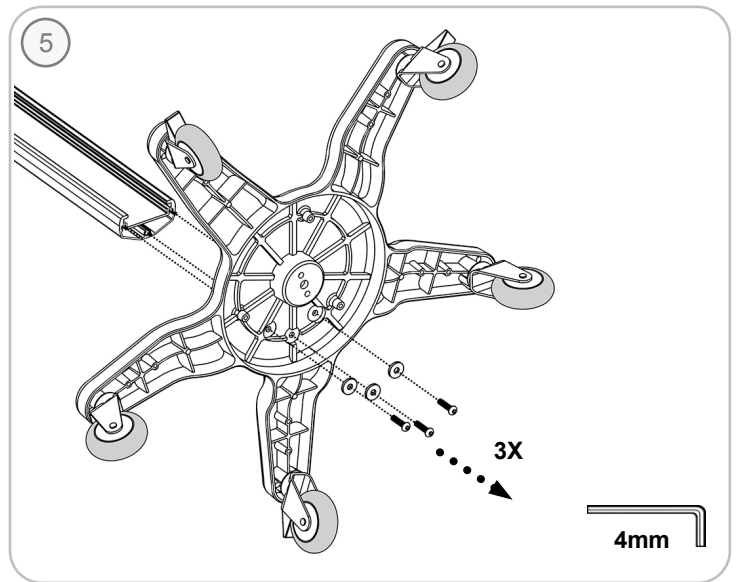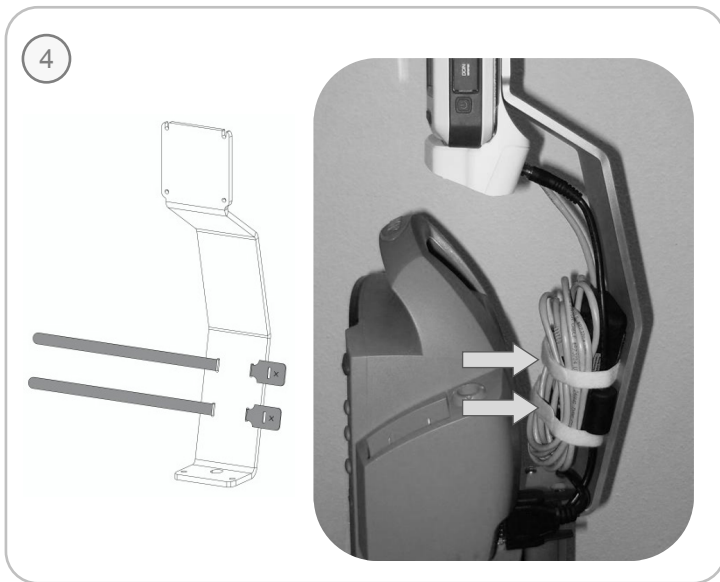

GCX

Mounting Solutions

gcx.com | (800) 228-2555 | +1 (707) 773-1100 | Fax +1 (707) 773-1180

GCX European Office | +31(0) 88 627 2600 | Fax +31 (0) 88 627 2609

GCX Taiwan | (886) 2 2298 2842 | Fax (886) 2 8990 3052



This roll stand and its accessories may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment. The below listed agents are known to be compatible with this product. This list is not exhaustive and other non-listed agents may also be found compatible for use.

Water
Mild Soap Solutions
Hydrogen Peroxide Solution, 3%
Iodine Solutions
Bleach Solutions, 10%
Isopropyl Alcohol Solutions, 70%, 91%
QUAT Disinfectants
Cavicide®
Cidex OPA®
Cidex Plus®
Formula 409®
Kleenaseptic®
Lysol® Disinfectant
Sagrotan®
Sani-Cloth® HB
Sporicidin®
Surfanios
Vesphene®
Virex® II 256



The surface finish of this roll stand and its accessories may be permanently damaged by strong chemicals, solvents or abrasives. These listed materials are known to cause probable damage to this product and should not be used. This list is not exhaustive and other non-listed agents may also be found non-suitable for use.

Acetone
Trichloroethylene
Steel wool or other abrasive material
Ethanol 100%
Oxivir® TB

Never submerge the product.

Damage caused by the use of unapproved substances or processes will not be warranted.